

NEW

One-Stop™

Fast and simple gingival retraction and hemostasis with aluminum chloride

Retraction Bleeding Control in One Step



Less time, better 2 in 1 effectiveness

Rapid and reliable bleeding control through the astringent action of aluminum chloride

Uniform paste application ensures excellent mucosal adhesion and dispersion

Comfort for both clinicians and patients

Reduced discomfort, fast and clean rinsing

Two viscosities and shades to support treatment strategies

MDCLUS
MEDICLUS CO., LTD.

One-Stop™

2 in 1 mechanism delivers enhanced precision for superior clinical outcomes.

One-Stop™ is a paste-type topical pre-impression agent that provides both gingival retraction and bleeding control simultaneously. With the astringent effect of aluminum chloride, it ensures rapid and stable bleeding control, while its uniform paste consistency allows precise application even in delicate areas. After use, it can be easily rinsed away with water, improving clinical efficiency and enhancing patient comfort.

Indications

- Gingival retraction and hemostasis prior to impression taking
- Bleeding control during crown and bridge preparations
- Moisture and bleeding management before bonding or cementation
- Gingival management for Class II and V restorations
- Temporary gingival displacement to improve clinical visibility


• Medium Flow (Yellow)

- For shallow to medium gingival sulcus
- Expansion and drying of the sulcus prior to digital or conventional impression taking
- Suitable for temporary bleeding control

• Low Flow (Blue)

- For deep sulcus or thick gingival tissue
- Recommended as a substitute for the second cord in the double-cord technique

Step by Step Process

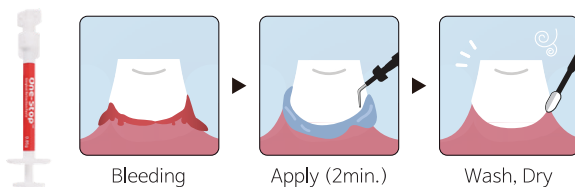

Clinical Case by Dr. Min-Sik Choi, Seoul Smart Dental Clinic




Delivery Forms



- #1101-1 One-Stop™ Blue 0.85g X 7ea
- #1101-2 One-Stop™ Blue 0.85g X 2ea
- #1102-1 One-Stop™ Yellow 0.85g X 7ea
- #1102-2 One-Stop™ Yellow 0.85g X 2ea

Color

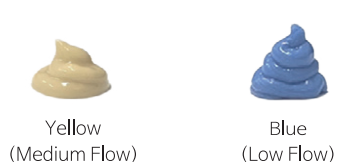

MEDICLUS

Headquarter | No.1210, 134, Gongdan-ro, Heungdeok-gu, Cheongju-si, Chungcheongbuk-do, 28576, Korea | +82 43 211 2877
Sales Office | No.206 Brown Stone, 464, Cheongpa-ro, Jung-gu, Seoul, 04510, Korea | +82 10 8385 2877
www.mediclus.co.kr | sales@mdclus.com

